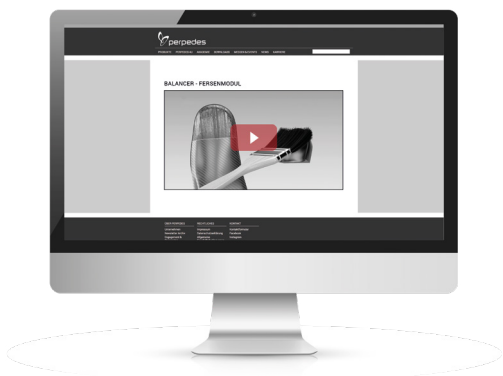
# BALANCER - FERSENMODUL

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EN – PROCESSING INSTRUCTIONS – BALANCER HEEL POST MODULE  
FR – INSTRUCTIONS D'UTILISATION – BALANCER - MODULE DE TALON  
ES – INSTRUCCIONES BALANCER - TALONES  
NL – GEBRUIKSAANWIJZING BALANCER - HIELMODULE

DE – VIDEOANLEITUNG  
EN – VIDEO TUTORIAL  
FR – TUTORIEL VIDÉO  
ES – TUTORIAL DE VÍDEO  
NL – VIDEO TUTORIAL

DEUTSCH

S.4

ENGLISH

S.16

FRANÇAIS

S.28

ESPAÑOL

S.40

NEDERLANDS

S.52

## INHALT

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## CONTENTS

Introduction .....	18
Fields of application and intended use .....	19
Contraindications .....	19
General information for use and safety .....	20
Guarantee .....	21
Instructions for set-up and processing .....	22
Customisation .....	24
User instructions for cleaning and disinfection .....	26
Technical data .....	27

**For visually impaired people, this document can be downloaded as a PDF on [www.perpedes.de](http://www.perpedes.de). IT IS ALSO AVAILABLE IN LARGE PRINT.**

# INTRODUCTION

Dear clients,

The present processing instructions for use are applicable for the **BALANCER** heel post module by Perpedes for attachment to Perpedes' foot orthoses.

The Perpedes management system is certified as per DIN EN ISO 9001 and ISO 13485.

If you have any questions on the use, maintenance and safety of this product, please contact Perpedes in writing or by phone.

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T. +49 (0)7021 738 30-0  
info@perpedes.de  
www.perpedes.de

**IMPORTANT NOTICE: PLEASE ONLY USE THIS PRODUCT AFTER HAVING FULLY READ AND UNDERSTOOD THE PRESENT INSTRUCTIONS FOR USE.**

## FIELDS OF APPLICATION AND INTENDED USE

The BALANCER heel post module has been designed for correction of foot/leg deformities. Possible indications are e.g. difference in leg length, varus / valgus deformities of the heel, afflictions of the Achilles tendons, wear (arthrosis) of the ankle, knee or hip joints. It is suitable for use by children, adolescents and adults.

The BALANCER has been developed for use in combination with an orthopedic arch support in shoes with closed heel section (boots, low shoes, work shoes, and the like). Only in this way can the efficiency of the product be guaranteed. The product must only be used for the intended purpose and under the specified conditions.

**Prior to initial use, the BALANCER must be adjusted by qualified experts to match the demands of the user. If this is not done, product adherence cannot be guaranteed.**

The expected service life of the BALANCER is 1 year.

## CONTRAINDICATIONS

Please take into account the following indications in your customers / patients:

- Diseases or injuries of the skin in the area of application
- Signs of inflammation, such as redness, warming or swelling
- Circulatory problems
- Disturbances in the lymphatic drainage
- Sensory problems in the area of application

In case that you identify any unusual changes or modifications, please inform the customer / patient to promptly seek medical advice.

## GENERAL INFORMATION FOR USE AND SAFETY

The technology and design of the BALANCER have been developed for maximum safety. The currently applicable international safety standards have been met and partly exceeded. However, the user may be exposed to risks in case that the product is not handled correctly. For your own safety, we urge you to read and always heed the rules and specifications listed below. Unprofessional or incorrect changes at the product will increase the risk of damage.

- The product is exclusively intended for compensation of foot / leg deformities. Any other use does not comply with its actual purpose.
- The BALANCER must only be worn in accordance with the specifications of the present instructions and the stated examples of use.
- In case that the available inner volume of the shoe is not sufficient for the foot plus the selected orthotic support incl. BALANCER, fit problems (tightness, feeling of pressure, chafing) cannot be ruled out.
- The product is intended as unique and customized product for one user.
- Use of the BALANCER for activities with increased physical effort, e.g. sports, must be previously clarified and agreed with the attending physician or other medical experts.
- In case that the BALANCER heel post module gets damaged, please have it promptly replaced.
- Without prior approval, no additional products that are not in compliance with the intended use or that may change the structure of the BALANCER must be attached. Any combinations with other medical products that are not described in the instructions for use are subject to approval by Perpedes.
- The BALANCER must not be exposed to temperatures above 41° C (heating / radiators, direct sunlight, open fire).
- Inform your customers that they must always make sure that the BALANCER incl. arch support is correctly inserted in the shoe, and that it fits perfectly around the heel.

- The change in body statics due to the BALANCER strongly depends on the customization of the BALANCER by the medically qualified experts. Therefore, please make sure to familiarize your customers with the new body sensation during the break-in period.

## GUARANTEE

### GUARANTEE CONDITIONS\*

Repair or replacement shall be carried out by authorized Perpedes retail partners.

Notice: This guarantee is non-transferrable.

1. The guarantee conditions specified above are applicable for all products that have been purchased for the full retail price.
2. Perpedes generally does not assume liability, if the BALANCER needs to be replaced for one of the following reasons:
  - a. The product was not used, handled and cared for as specified in the manufacturer's recommendations.
  - b. The BALANCER was damaged due to negligence, accident or improper use.
  - c. Changes have been performed at the BALANCER that deviate from the specifications of the manufacturer.

\*THIS DOES NOT LIMIT YOUR LEGAL RIGHTS IN ANY WAY.

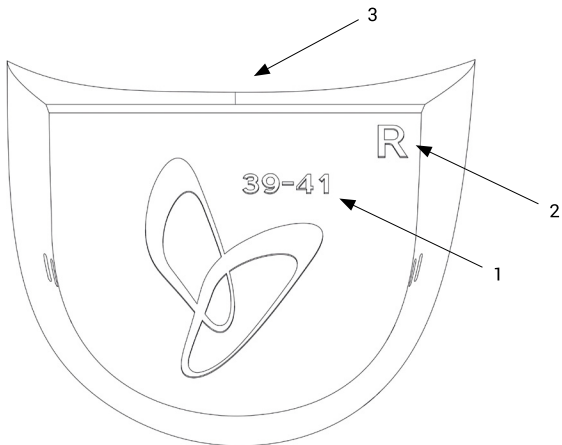
# INSTRUCTIONS FOR PROCESSING

## ADJUSTMENT TO FOOT ORTHOSES

The BALANCER must only be processed by the method described below in compliance with the defined product structure and instructions for adjustment.

To adapt the BALANCER to foot orthotics, please perform the following steps:

1. Take the BALANCER of the right size out of the packaging. The recommended size range of the orthopedic arch supports is marked at the bottom of the BALANCER **(1)**.
2. Place the BALANCER with its upper side onto the heel section of the orthotic's bottom side and mark the required position at the front edge and in the middle **(3)**. Make sure to use the module for the correct side **(2)** and that it fully covers the back part of the the orthotic's heel section!

3. Clean the top side of the BALANCER and the marked heel area to be free from grease and dust and apply commercially available contact-adhesive suitable for the TPU material. Then, leave to dry for 15 minutes. Please comply with the processing instructions by the adhesive manufacturer, particularly on the drying time of the adhesive.
4. Place the BALANCER onto the orthotic's base in the marked position and firmly press it on.

## CUSTOMIZATION

Due to its design, the BALANCER can be adapted to a variety of conditions:

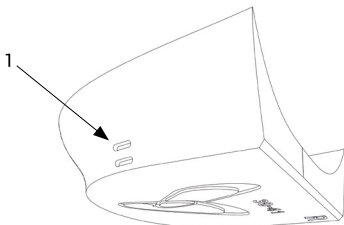
**NEUTRAL POSTING, DIFFERENCE IN LEG LENGTH, AFFLICTIONS OF THE ACHILLES' TENDON:**

NEUTRAL


The BALANCER can be adjusted to compensate a leg length discrepancy of max. 4 mm. If smaller differences are to be compensated, the BALANCER needs to be ground down evenly, both medially and laterally.

To simplify this, indicators **(1)** were laterally and medially cut into the BALANCER, which are intended to make processing of the material easier. The upper edge of the bottom indicator marks a reduction of 2 mm and the upper edge of the upper indicator marks a reduction of 4 mm.

## EVERSION / INVERSION POSTING, VARUS / VALGUS MALPOSITION OF THE HEEL, POSTURE-RELATED COMPLAINTS IN THE ANKLE, KNEE AND HIP JOINTS:

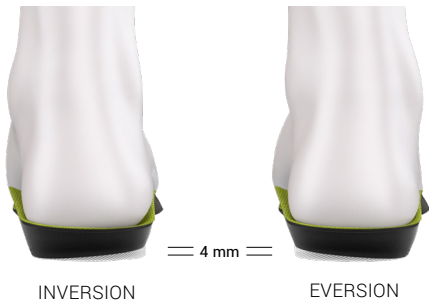With the help of the BALANCER, the position of the heel is corrected in regards with inversion or eversion, so that possible complaints of the foot / ankle / knee / hip can be positively affected. For this purpose, the BALANCER must be ground down either medially or laterally on one side more than on the other (depending on the required result). In this way precise correction of the physical position can be achieved. The medial and lateral indicators in the material help achieving this. The upper edge of the bottom indicator marks a reduction of 2 mm and the upper edge of the upper indicator marks a reduction of 4 mm.

During processing, please make sure that the bottom surface of the BALANCER forms full and even contact with the supporting structure when viewed from the side and that its stable positioning in the shoe is ensured. If this is not the case, the BALANCER needs to be reworked.

After rough grinding, the lateral surfaces of the BALANCER must be finished using fine grinding belt and ideally also a polishing brush.

## USER INSTRUCTIONS FOR CLEANING AND DISINFECTION

Check the attached BALANCER for wear and tear every 4 weeks.

If required, clean the BALANCER using a mild cleaning agent and a soft sponge or cloth. After cleaning, the soap residues must be thoroughly removed with water. The BALANCER is not suited for cleaning in the washing machine.

Do not use external heat sources (radiator, blower, direct sun light) for drying but leave it to air dry.

Mild disinfectants that are suitable for contact with human skin may be used (sprayed on) in small amounts. However, in such cases discoloration of the materials may occur.

**CAUTION!** The BALANCER may be damaged due to sand and sea water. Thoroughly clean the BALANCER after it was exposed to such conditions.

### DISPOSAL

You can dispose of the product in accordance with the applicable national rules and regulations.

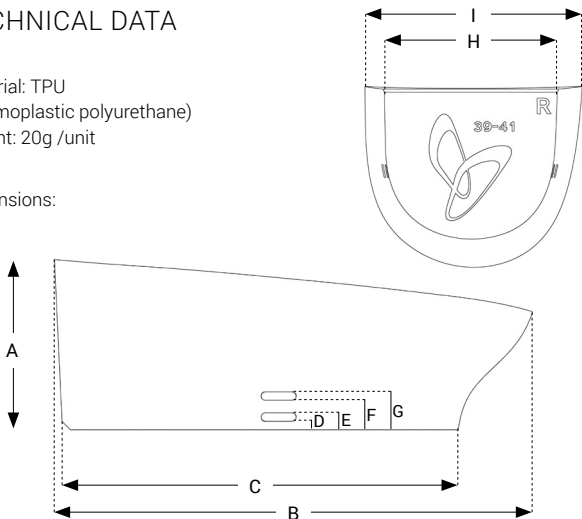
### STORAGE CONDITIONS

The products must not be exposed to direct UV light and must be stored at a temperature range between 10 °C and 25 °C without any mechanical influences. Regular control of the stored foot orthoses and of the storage conditions is recommended by the manufacturer.

## TECHNICAL DATA

Material: TPU  
(thermoplastic polyurethane)  
Weight: 20g /unit

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Perpedes GmbH